

HOW TO DOWNLOAD OFFICE MEDIC WITH WINDOWS 10

Step 1. Open your web browser, type vectracor.com/software-download/ in your search bar.

Step 2. You will be directed to the **Software Download** page. **Scroll Down** and **click Download Now** under the **Office Medic** section.

*** For Windows 7 Please click “How to Activate Windows 7” Before continuing. These instructions are ONLY for Windows 10 since Microsoft customer service is no longer available to provide Windows 7 technical support.**

Step 3. You will be directed to the **Log In to Your Account** page. **Enter the Login/License ID and Login/License Password provided on the back of this page (Not your email).**

***Please note that the Login ID/License ID and Login/License Password are case sensitive. Disregard Existing Customer.**

Email or Login ID:

Password:

[Forgot your password?](#)

Step 4. You will be directed to the **Customer Information** page. **Fill out your facility information of your facility information, click Submit** when finished.

Step 5. You will be directed to the **License Portal Home** page. Under the **Welcome bar, click Downloads.**



Step 6. You will be directed to the **Downloads** page. **Click Download** to continue.


Step 7. You will be directed to the [Box Website](#). **Hover your mouse over to the very right of Windows 10 VECG & Office Medic Installer.iso** and **Click the three dots** that appear with the tag pop up **More Options**. **Click the Download** button that drops down.

VECG408_OM405_0128_2020


Name	Updated	Size	
Windows 7 VECG 4.0.8.1 Installer	Yesterday by Vincent	162 Files	
Windows 10 VECG 4.0.8.1 Installer.iso	Feb 3, 2020 by Vincent	411.5 MB	...

Download

Step 8. The **Your Download is In Progress** window will appear, **no action is required, Click X.**

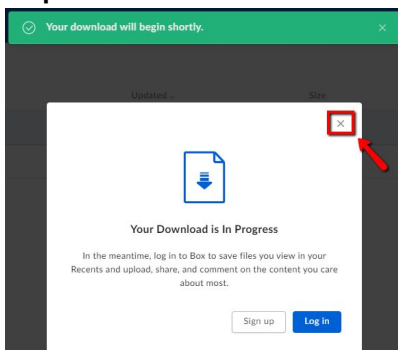


Step 9. Your **Installer** will pop up on the **top** or **bottom** of your window, *depending on your browser*. **Double Click the Installer, WAIT TILL IT'S DOWNLOADED** to continue.

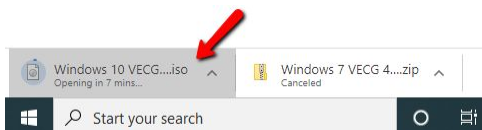


Step 10. Your **Files** window will appear. **Scroll down and Click DVD Drive...** **Scroll down, locate, and Double Click Setup.**


**There will be several Setups, ensure that the one you select matches the Image.*

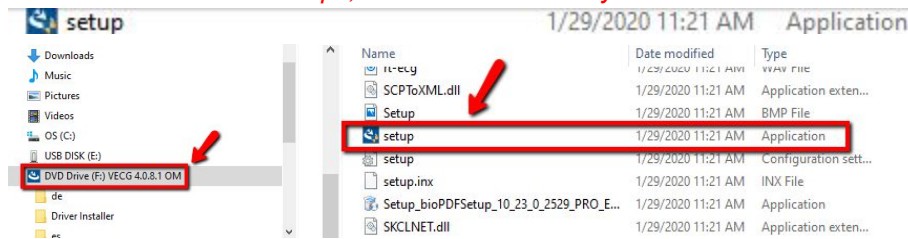


Step 10. Your Files window will appear. Locate and Click the Extract All Files Tab.

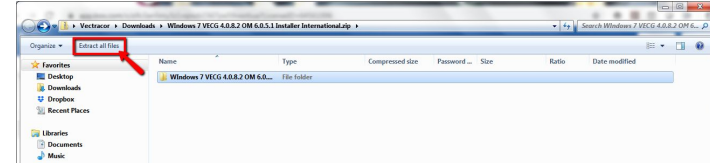

Step 10a. An Extract Compressed (Zipped) Folders Window will appear, click Extract.

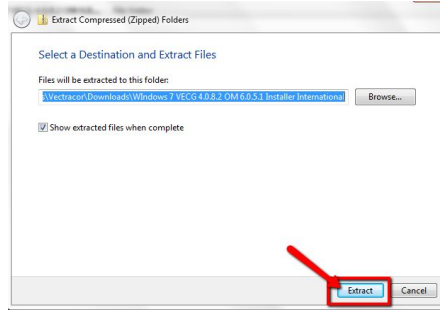


Step 10b. Your Files window will appear. Click Windows 7 VEGC

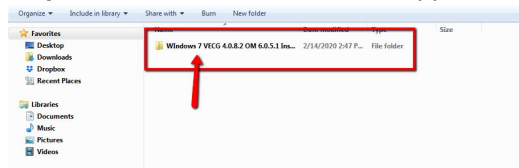

Step 10c. Your Files window will appear. Scroll down, locate, and Double Click Setup.



**There will be several Setups, ensure that the one you select matches the Image.*

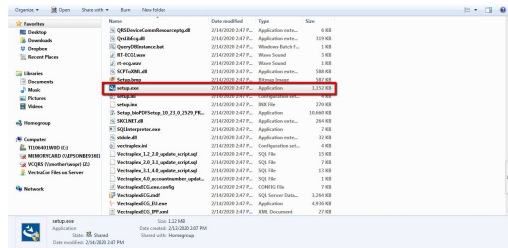

Step 11. The User Account Control window will appear, click Yes when asked "Do you want to allow this app from VectraCor to make changes to your device". **Image not shown*

Step 12. The Choose setup language window will appear. Select your language of choice and Click Next.

Step 13. The Welcome to the InstallShield Wizard window will appear. Click Next.


Step 14. The Setup Type window will appear, select VetraplexECG and Office Medic and Click Next.

Step 15. The Choose Destination Location window will appear, Click Next.

Step 16. The Set Program Options window will appear, click Local Database or click Network Database depending on your preferences, Click Next.

Step 17. The Please Choose Defaults window will appear, click United States of America or International, Click Next.

Step 18. The Setup Status window will appear, Wait for download, Click Next.

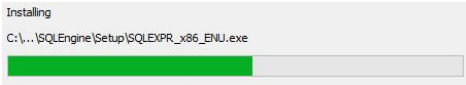

Step 19. The **Welcome to the Device Driver Installation Wizard!** window will appear, **Click Next.**

Please disregard the windows of code behind your Installation Wizard windows, **do not close them out.*

Step 20. The **Completing the Device Driver Installation Wizard** window will appear, after the checkmarks under **Driver Name** turn green and the **Status** states **Ready to use**, **Click Finish.** **Please disregard the windows of code behind your Installation Wizard windows, **do not close them out.***

Driver Name	Status
✓ VectraCor, Inc Portable ...	Ready to use
✓ VectraCor, Inc Universal...	Ready to use
✓ VectraCor, Inc Universal...	Ready to use

Step 21. The **Installshield Wizard Complete** window will appear, **Click Finish.**

Step 22. Close out **VectraplexECG** and open up **Office Medic** to test your new software to ensure proper download.

** If the program is not loading please contact our customer support team at QRSsupport@vectracor.com, or give us a call at (973) 904-0444 Ext. 2 *PLEASE DO NOT THROW OUT THIS PAPER, IT IS THE ONLY PLACE YOU CAN ACCESS YOUR ID AND PASSWORD.*