

## Hot Tips Report

---

<b>ID</b>	0036	<b>Description</b>	Sub or Function Not Defined
<b>Date</b>	8/15/2005 2:51:27 PM		
<b>Manager</b>	ADMIN	<b>Status</b>	Active

**Text**      **Troubleshooting "Sub or function not defined" error**

**Problem**

This "Sub or function not defined" error can be caused by old files remaining in folders on the C: drive.

**Solution**

1. Check to see if the old following files exist  
C:\WINDOWS\SYSTEM  
C:\WINNT\SYSTEM  
If the files exist continue on to step 2. If they do not exists skip to skip 3.
2. Delete files C:\WINDOWS\SYSTEM and C:\WINNT\SYSTEM
3. Ensure the new files have been successfully created and stored:  
C:\WINNT\SYSTEM\Mmecg200.vbx  
C:\WINNT\SYSTEM\Mmres100.dll  
C:\WINNT\SYSTEM\mmscp200.dll  
C:\WINNT\SYSTEM\Mmscpecg.dll

If after completing the above steps you have not solved the problem, contact Technical Support.

---