

Hot Tips Report

ID	0020	Description	TN 04.04 - Re-installing Office Medic v4.X
Date	4/26/2005 10:31:36 AM		
Manager	ADMIN	Status	Active

Text **Reinstallation of Office Medic**

The information in this Tech Note applies to:

- Office Medic v4.0 and higher.

Summary

This technical note explains the steps that may occur for an end user when they reinstall Office Medic version 4.0 or later.

If a user reinstalls Office Medic, there is the potential that some temporary installation files from the previous installation will not have been deleted. Thus, the system will prompt the user indicating that the file will be overwritten. The user may choose to 1) not overwrite the file, 2) overwrite it, 3) overwrite all, 4) not overwrite all, or 5) cancel the installation.

Solution

The user should select the 'overwrite all' option.
