



V300 Holter Gold Cardiac Analysis Software Installation Instructions

1. Open your web browser and navigate to: www.linktr.ee/V300Gold
2. Click on “Software Installation”, it will take you to the software download link.

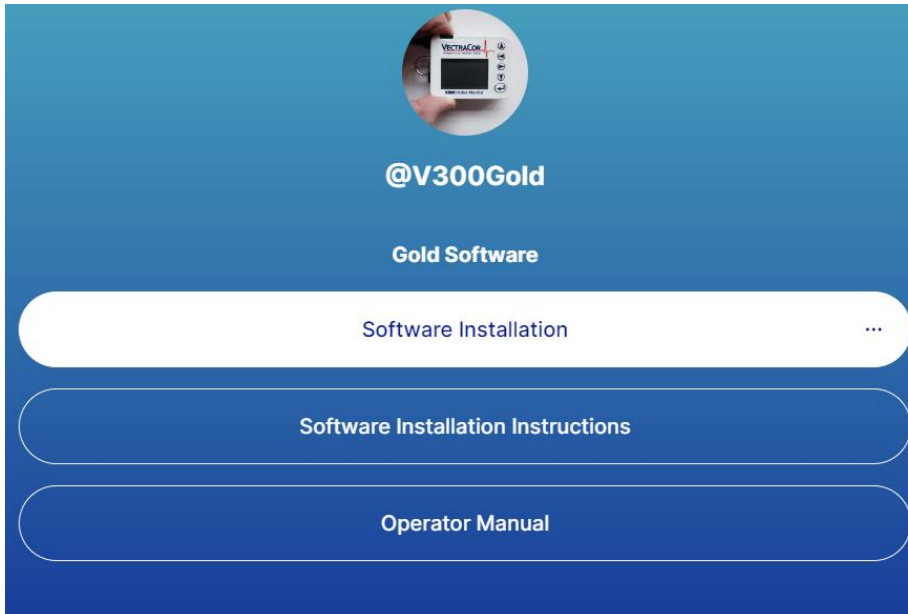





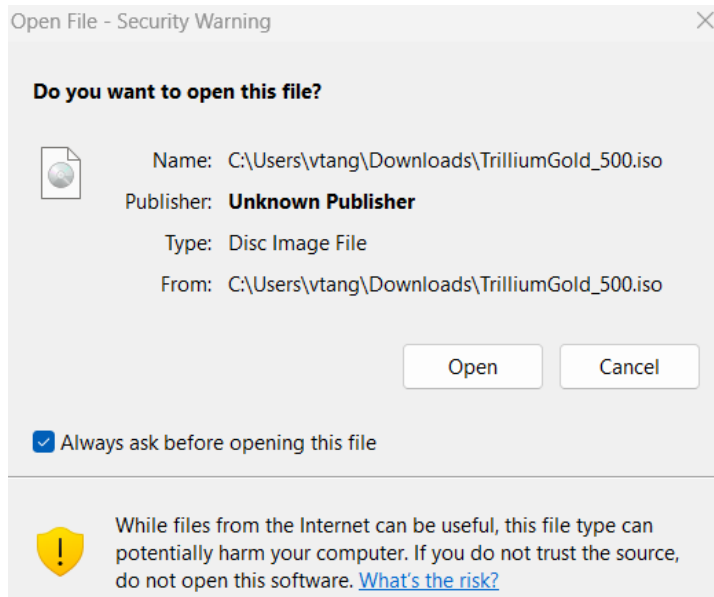





3. Click on the file “TrilliumGold_500.iso”
4. Click download on the top right.


5. Open the file and click “Open” when prompted with this window:




6. Find the “setup” application, right click and run as administrator. (Make sure you are opening the application and not the configuration settings.)

	Holter Data	11/30/2023 3:19 PM	File folder
	program files	11/30/2023 3:19 PM	File folder
	System32	11/30/2023 3:20 PM	File folder
	Win	11/30/2023 3:20 PM	File folder
	0x0409	11/2/2023 6:19 PM	Configuration setti...
	Autorun	11/2/2023 6:19 PM	Setup Information
	instmsiw	11/2/2023 6:19 PM	Application
	setup	11/2/2023 6:19 PM	Application
	Setup	11/2/2023 6:19 PM	Configuration setti...
	Trillium Gold	11/2/2023 6:19 PM	Windows Installer ...


7. You may get an error message: Click “More info” → “Run anyway”

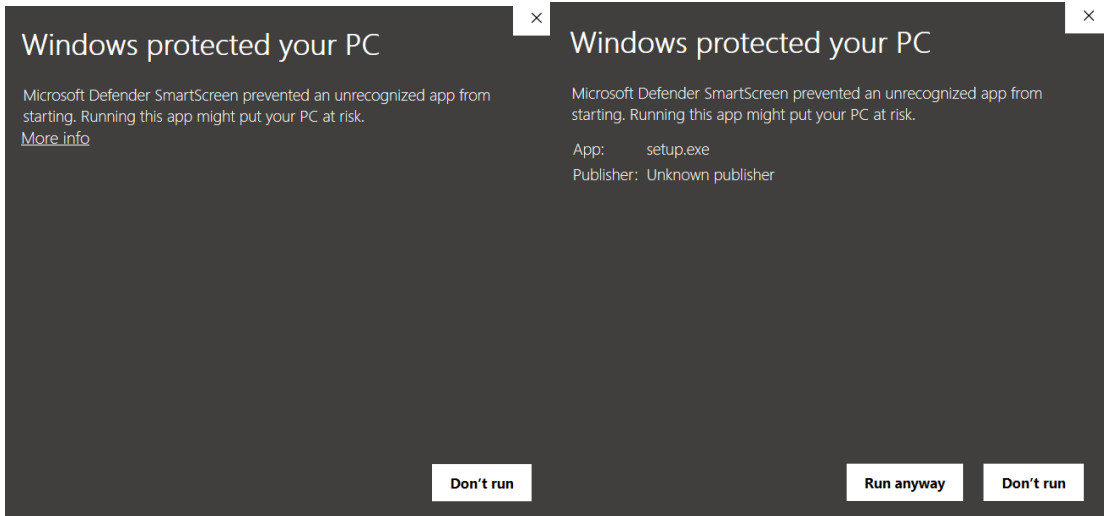

8. Click “Yes” when prompted with this window:

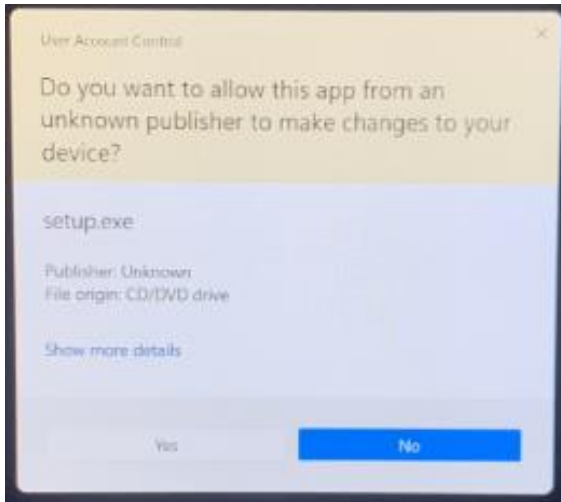


9. Click Next.

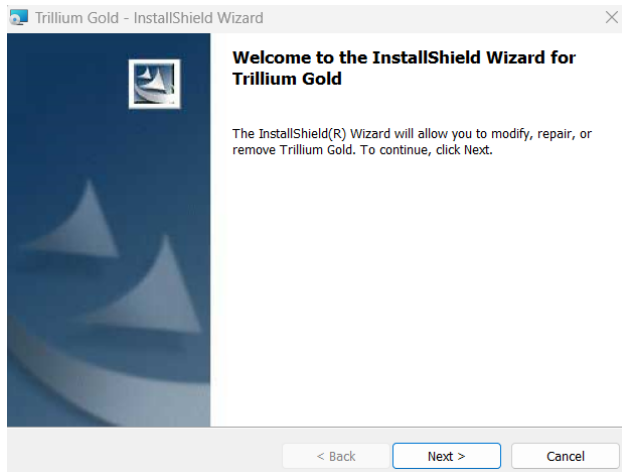

10. Choose whether the software is installed for a specific user account or the PC.

11. Click Finish

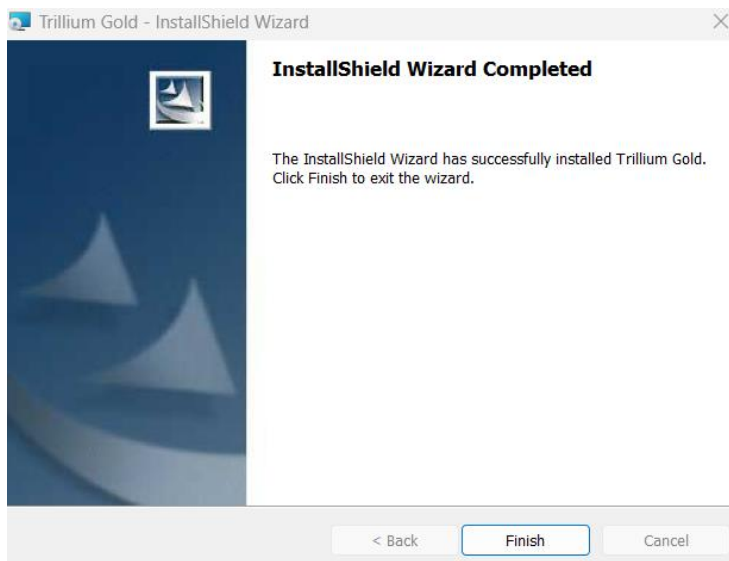

12. You should be able to find your newly downloaded software on the desktop.