



V300 Holter **Platinum** Cardiac Analysis Software Installation Instructions

1. Open your web browser and navigate to: www.linktr.ee/V300Platinum
2. Click on “Software Installation”

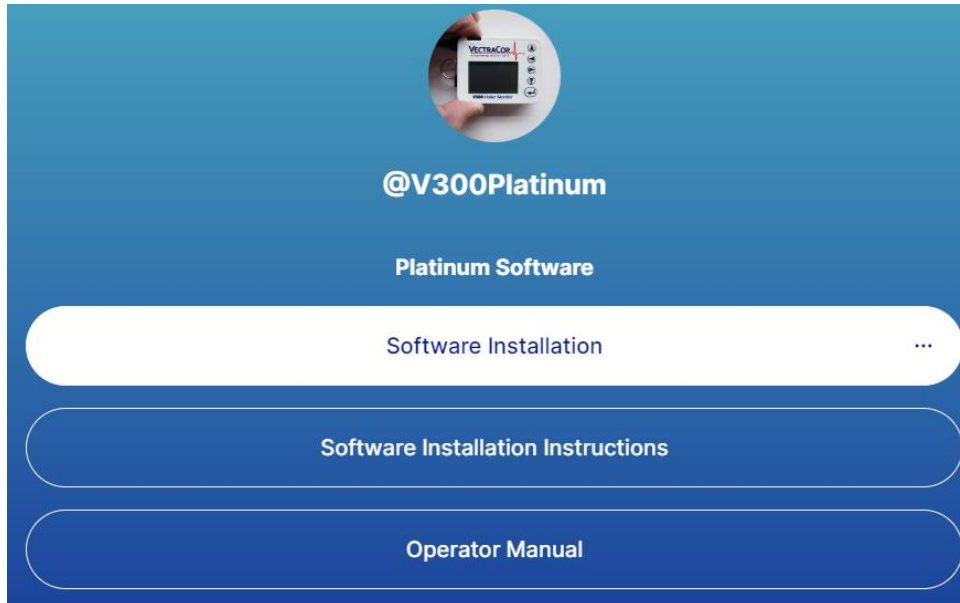


3. Click on the file “TrilliumPlatinum_500.iso”
4. Click download on the top right.
5. Open the file and click “Open” when prompted with this window:

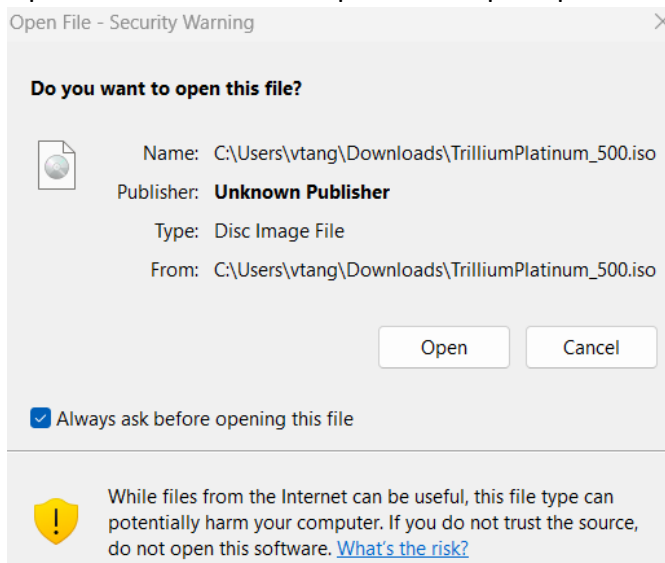


6. Find the “setup” application and run as administrator. (Make sure you are opening the application and not the configuration settings.)

📁 Holter Data	11/30/2023 3:22 PM	File folder
📁 program files	11/30/2023 3:23 PM	File folder
📁 System32	11/30/2023 3:24 PM	File folder
📁 Win	11/30/2023 3:24 PM	File folder
📄 0x0409	11/2/2023 11:57 AM	Configuration setti...
📄 Autorun	11/2/2023 11:57 AM	Setup Information
📄 instmsiw	11/2/2023 11:57 AM	Application
📄 setup	11/2/2023 11:57 AM	Application
📄 Setup	11/2/2023 11:57 AM	Configuration setti...
📄 Trillium Platinum	11/2/2023 11:57 AM	Windows Installer ...


7. You may get an error message: Click “More info” → “Run anyway”

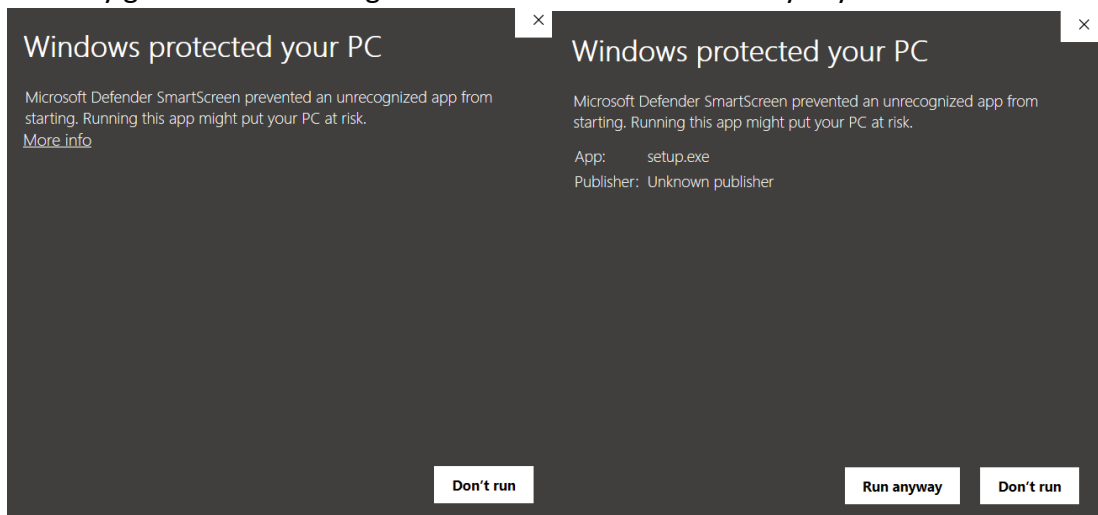

8. Click “Yes” when prompted with this window:

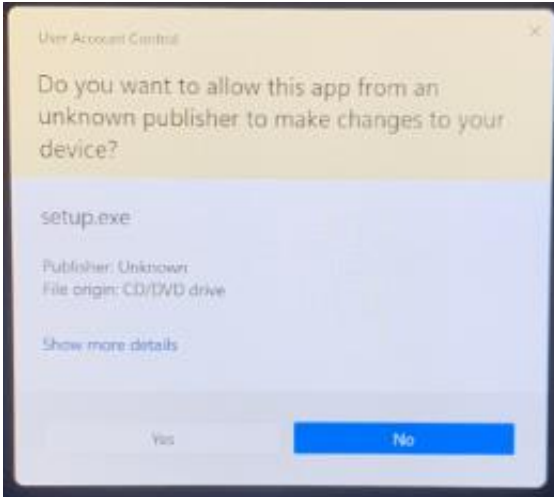


9. Click Next

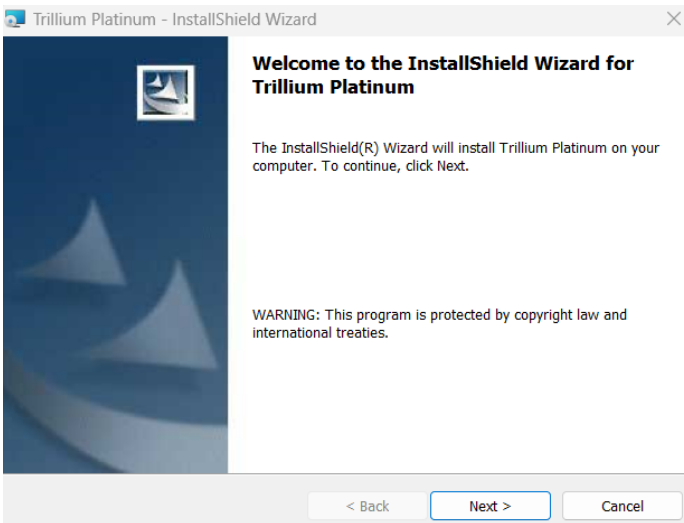

10. Choose whether the software is installed for a specific user account or the PC.

11. Click Finish.

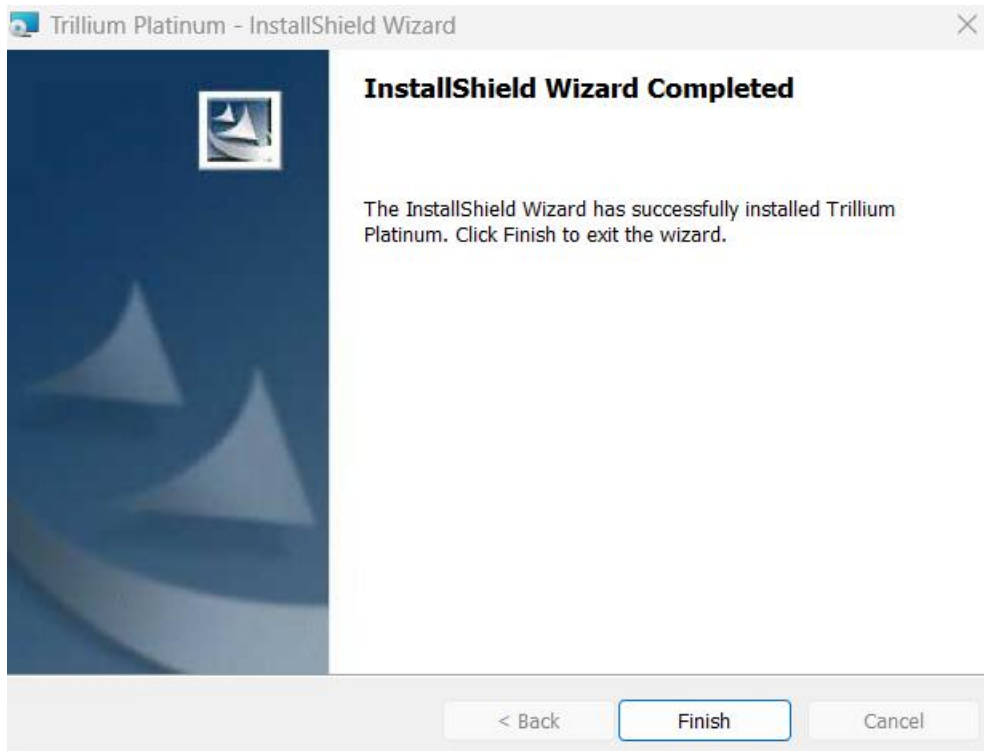

12. You should be able to find your newly downloaded software on the desktop.