



# V300 Holter Silver Cardiac Analysis Software Installation Instructions

---

1. Open your web browser and navigate to: [www.linktr.ee/V300Silver](http://www.linktr.ee/V300Silver)
2. Click on “Software Installation”, it will take you to the software download link.

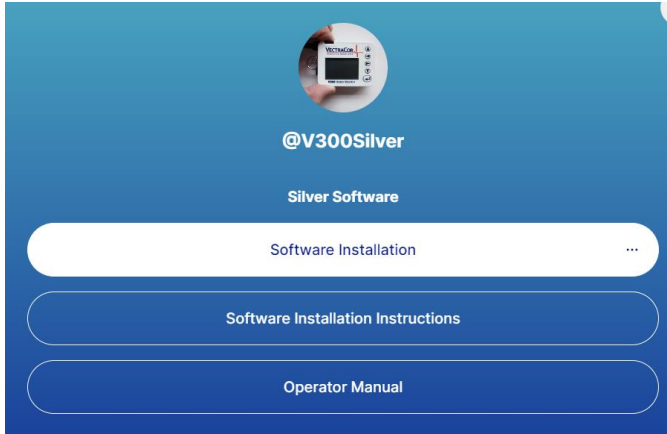


3. Click on the file titled “TrilliumSilver\_500.iso”
4. Click download on the top right
5. Open the file and click “Open” when prompted with this window:

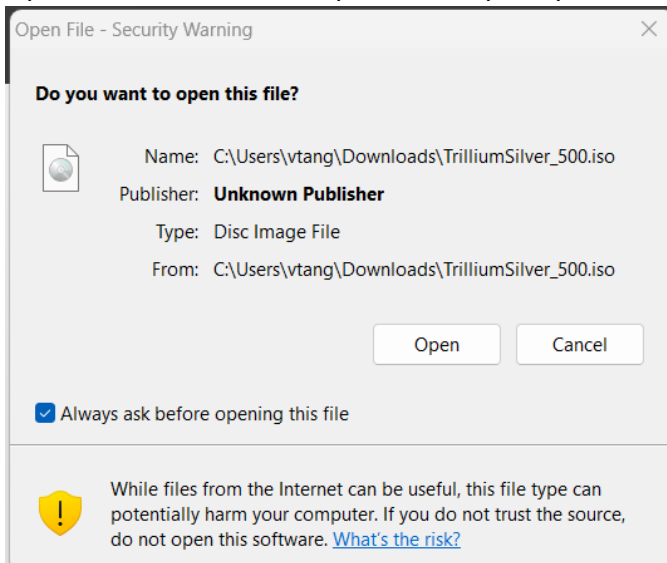


6. Find the “setup” application and run as administrator. (Make sure you are opening the application and not the configuration settings.)

Folder	Holter Data	11/30/2023 3:16 PM	File folder	
Folder	program files	11/30/2023 3:16 PM	File folder	
Folder	System32	11/30/2023 3:17 PM	File folder	
Folder	Win	11/30/2023 3:17 PM	File folder	
File	0x0409	11/2/2023 10:21 PM	Configuration setti...	22 KB
File	Autorun	11/2/2023 10:21 PM	Setup Information	1 KB
File	instmsiw	11/2/2023 10:21 PM	Application	1,780 KB
File	setup	11/2/2023 10:21 PM	Application	1,179 KB
File	Setup	11/2/2023 10:21 PM	Configuration setti...	6 KB
File	Trillium Silver	11/2/2023 10:21 PM	Windows Installer ...	838 KB


7. You may get an error message: Click “More info” → “Run anyway”

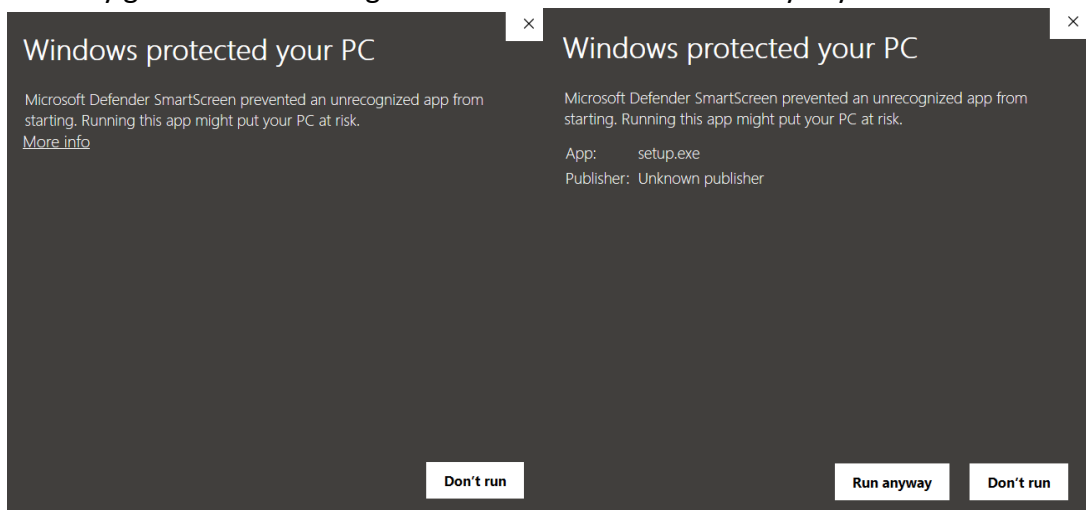

8. Click “Yes” when prompted with this window:

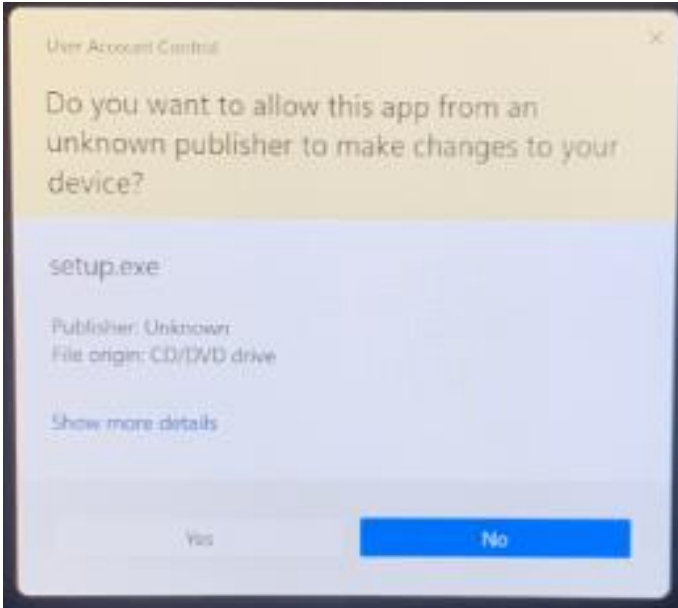


9. Click Next.

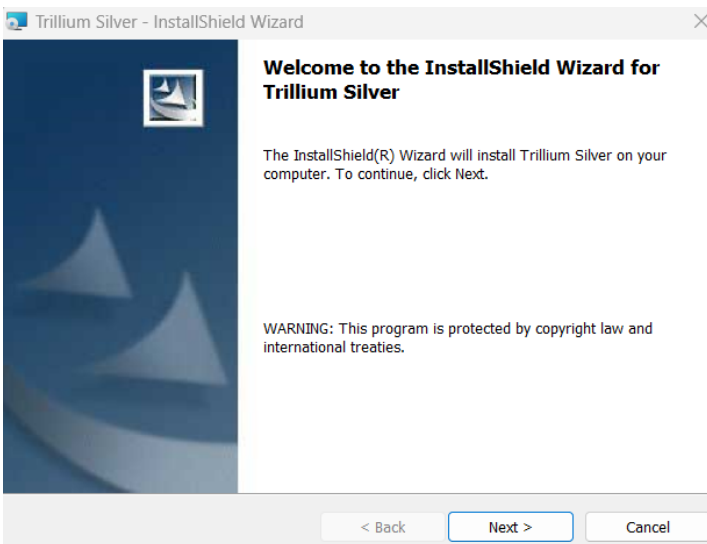

10. Choose whether the software is installed for a specific user account or the PC.

11. Click Finish

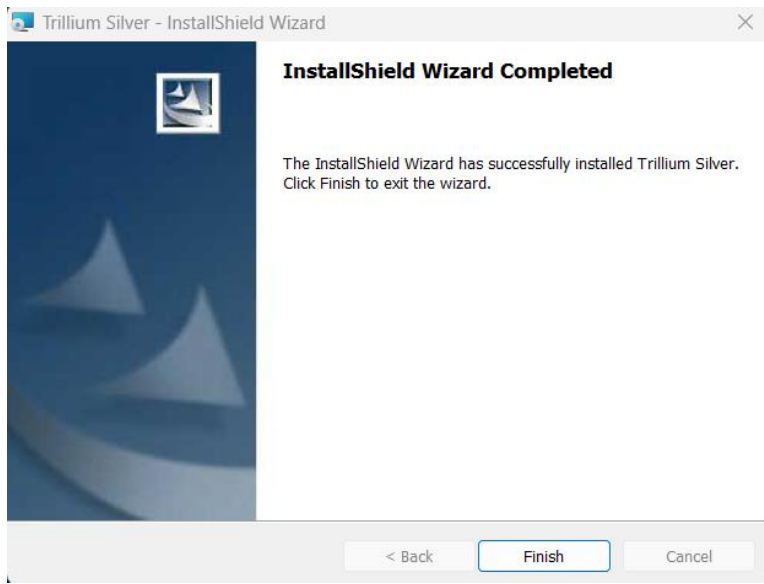

12. You should be able to find your newly downloaded software on the desktop.