


V300 Holter **Platinum** Cardiac Analysis Software Installation Instructions


1. Confirm that the PC you are trying to install the software on has administrative privileges.
2. Open your web browser and navigate to: www.linktr.ee/V300Platinum
3. Click on “Software Installation”.




4. Download the folder titled “V300Platinum v6.0”.
You can either:
 - a. Select the file (check the checkbox) and click the button which is located on the top right of the screen

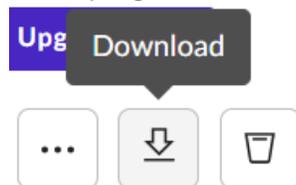


- b. Click the three dots and click “Download”

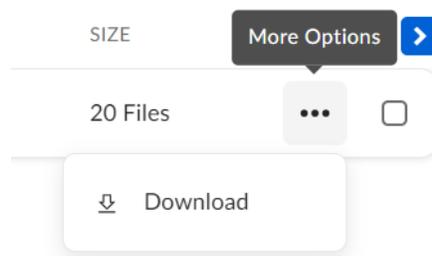

5. Right-click the folder named “V300Platinum v6.0.zip” and “Extract All...”

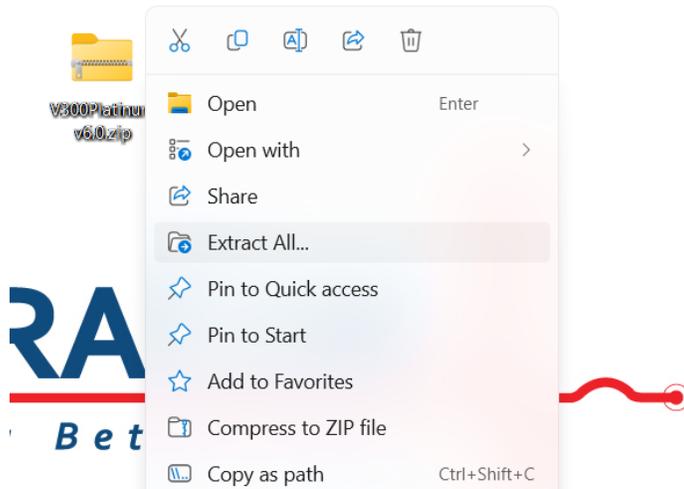


6. Open the newly extracted folder and locate the “setup.exe” application. Right-click the “setup.exe” application and run the program as an administrator.

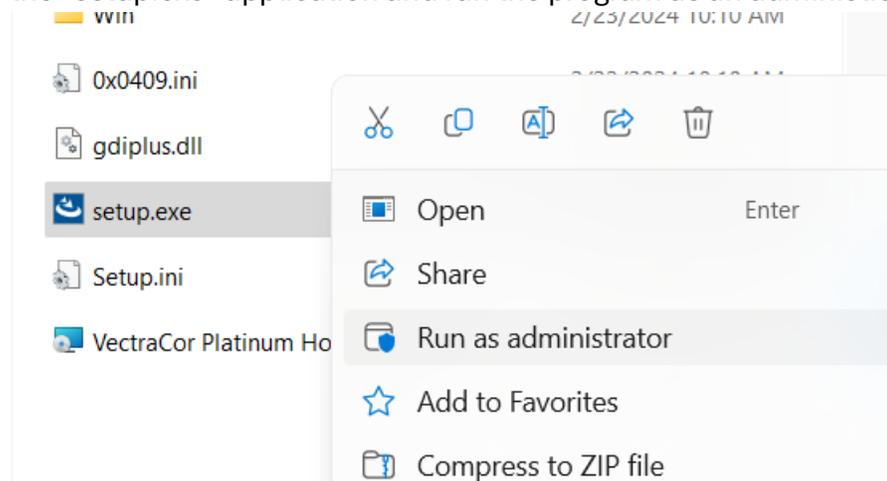


7. You may get a Windows Defender SmartScreen pop-up message: Click “More info”
→ “Run anyway”




8. Click “Yes” when prompted with this window:

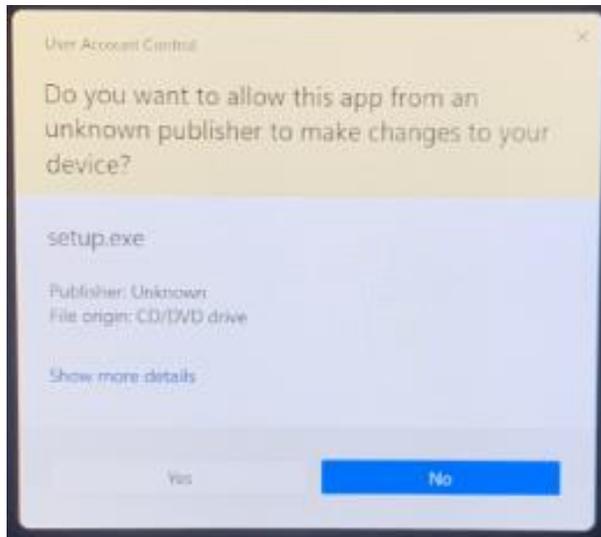


9. Install the KEYLOK Driver and select “KEYLOK2 (USB w/ Driver)” for the dongle type & “Standalone” as the installation type.

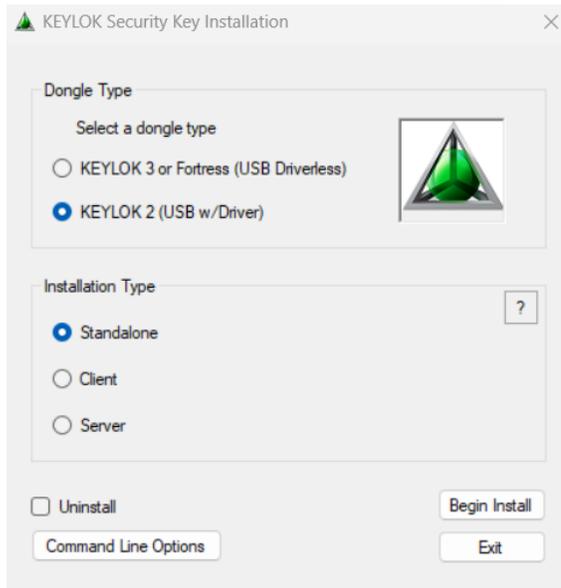


10. Complete the Device Driver Installation.

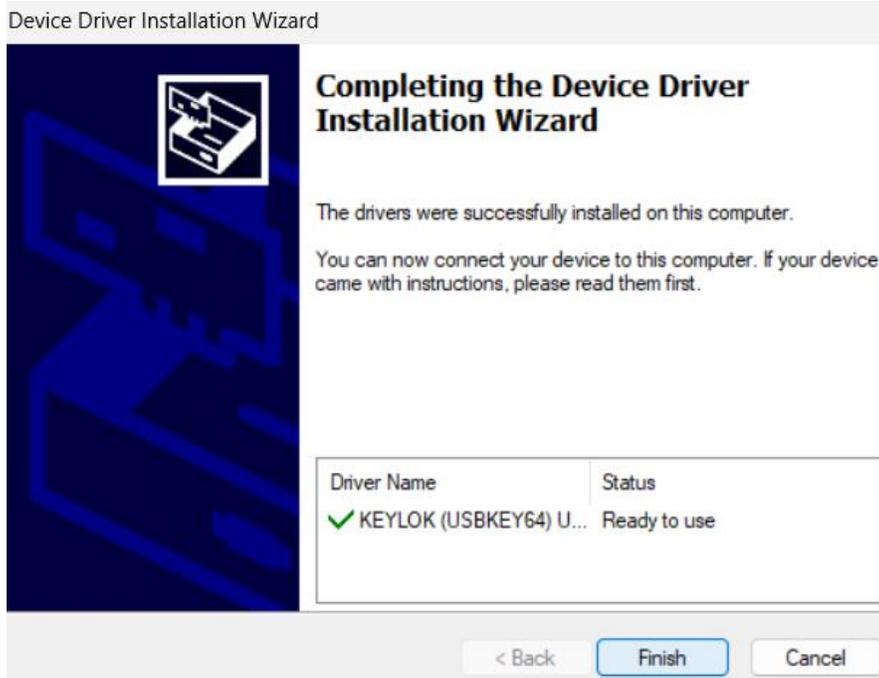

11. You may get the error message below; this is expected and will not affect the functionality of the Dongle or software.

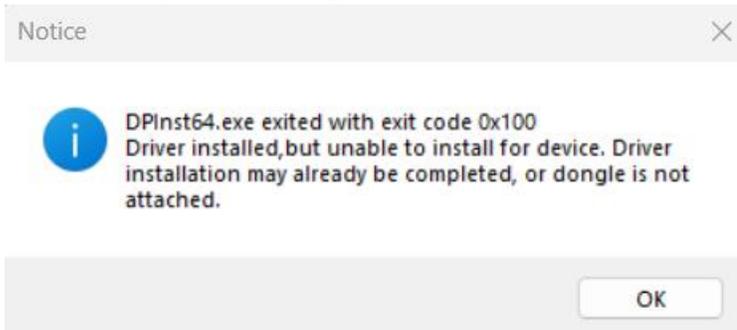

12. Read and accept the End User License Agreement.

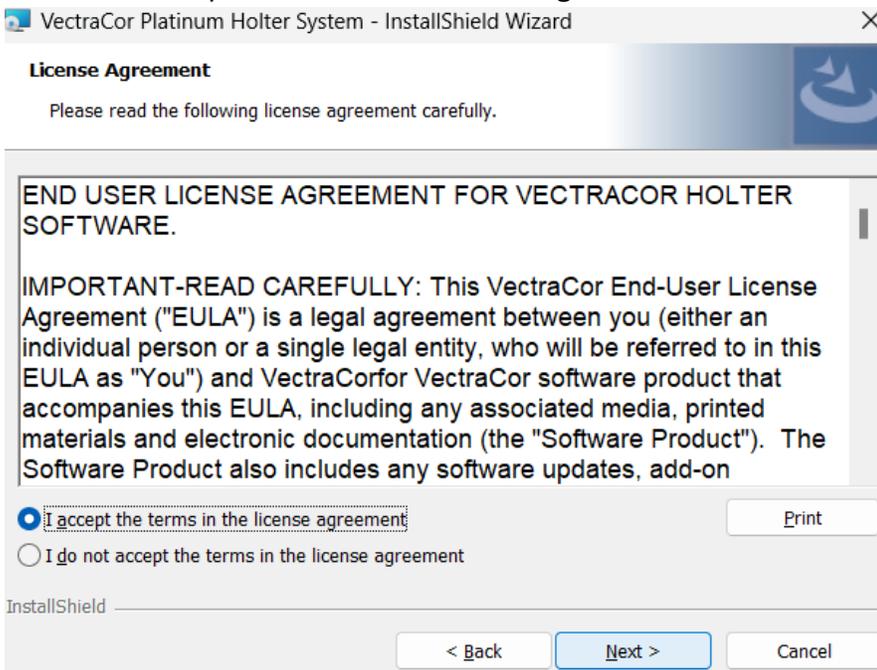

13. Enter in Customer Information.

VectraCor Platinum Holter System - InstallShield Wizard

Customer Information

Please enter your information.

User Name:
User Name

Organization:
VectraCor

InstallShield

< Back Next > Cancel

14. Choose a “Complete” installation for the setup type.

VectraCor Platinum Holter System - InstallShield Wizard

Setup Type

Choose the setup type that best suits your needs.

Please select a setup type.


Complete
All program features will be installed. (Requires the most disk space.)

Custom
Choose which program features you want installed and where they will be installed. Recommended for advanced users.

InstallShield

< Back Next > Cancel

15. Complete Software Installation

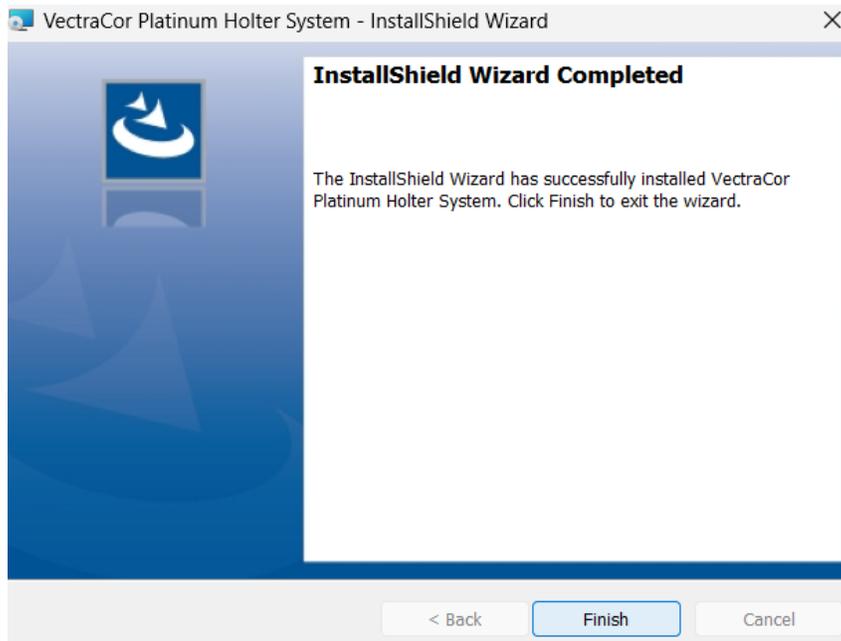

16. Navigate to the desktop for your software and for PDF files of the User Manual and Electrode Placement Guide,

