

FAQ: How to end V300 Holter recording

Normal Ending

When data acquisition time is completed, the device will automatically stop and power down. The screen will go blank – no data will be lost; it may then be returned to the office for transfer of the data.

Early Out

If the device has not turned itself off and is actively recording, the clinician can use a four key sequence to end the session immediately.

To do this first press \leftarrow <enter>, then \rightarrow <right> then \leftarrow <enter>, then the Event Button. Do not press the keys at the same time but do press them one after the other. If you have done so appropriately the device will prompt you with “Press EVENT” to end the recording. If you do not do so within 3 seconds, the key sequence is dismissed and the device continues to operate. This particular key sequence was selected with this specific timing to ensure that a patient bumping a device, rolling over on it in their sleep, or someone pressing the keypad at random would be unlikely to cause a premature shutdown of the device.

